


Trimalleolar Ankle Fracture Dislocation Treatment Using Plate and Screw Fixation

Case Study

Kent Ellington, MD

A 70-year-old female with a closed trimalleolar fracture dislocation was treated with the Acumed Ankle Plating System 3 and Small Fragment Base Set.

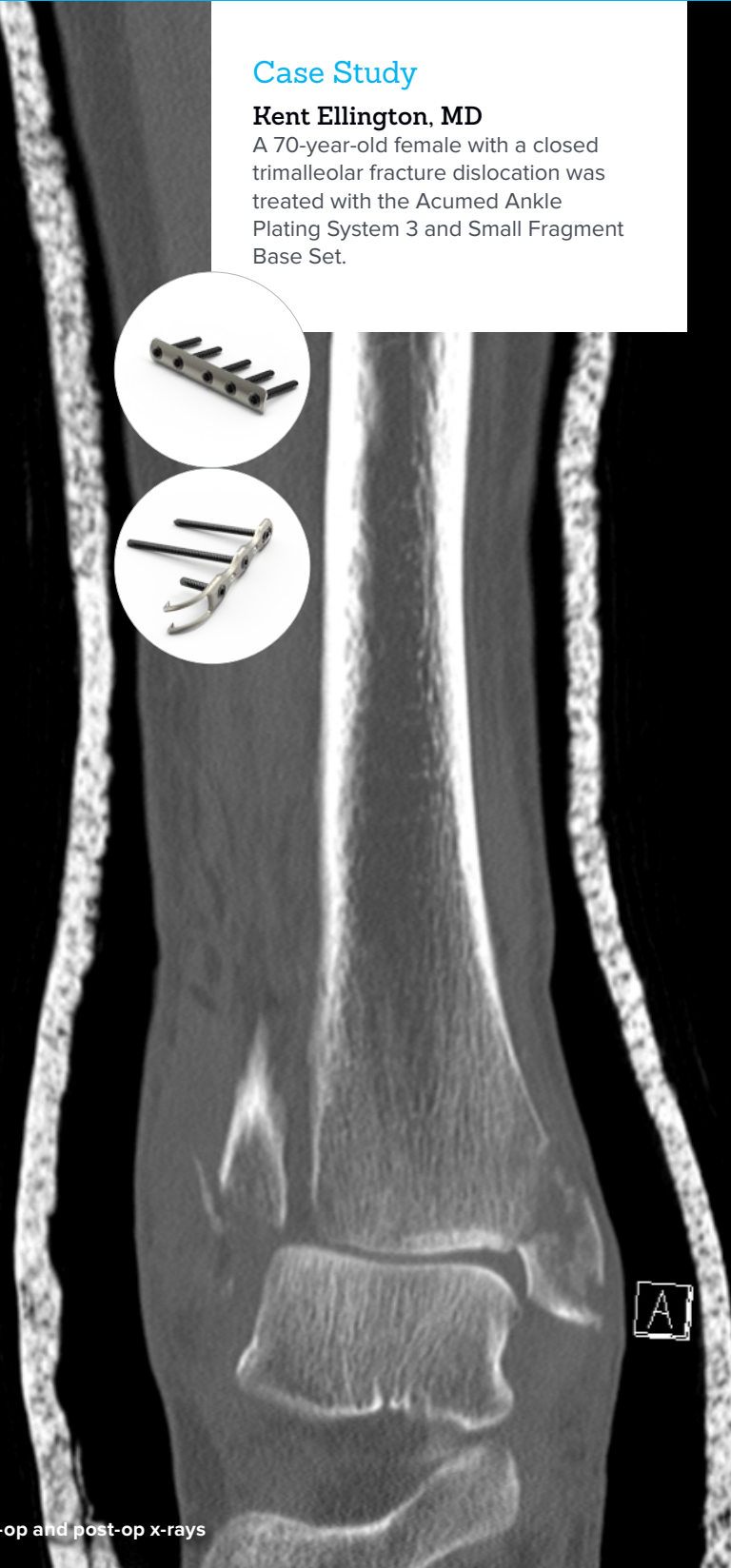

Acumed® is a global leader of innovative orthopaedic and medical solutions.



We are dedicated to developing products, service methods, and approaches that improve patient care.

Case Study | Kent Ellington, MD



Patient History

The patient is a 70-year-old female who fell off her bicycle in July 2015. She sustained a closed trimalleolar fracture dislocation that was reduced in the emergency room and splinted. A CT scan was obtained. The patient presented to the clinic four days later and went to the operating room on post-injury day eight.

Treatment

Treatment included open reduction internal fixation (ORIF) of the ankle fracture. The Acumed Ankle Plating System 3 and Small Fragment Base Set were used to provide fixation, which required a posterolateral approach to the ankle. The patient was placed in a prone position and the comminuted fibula was fixed with a bridge plate. Next, the posterior malleolus fragment was fixed with a buttress plate, then the comminuted medial malleolus was fixed with a Hook Plate.

Discussion

This fracture type requires a thorough understanding of the fracture mechanism, bone quality, and surgical approach. Ankle fractures present many variations. The necessary tools, instruments, plates, and screws required to complete ORIF of ankle fractures are all included in the Acumed Ankle Plating System 3.



8 weeks postoperative



LEX70-05-A | Effective: 2017/05 | © 2017 Acumed® LLC

Acumed® Headquarters
5885 NW Cornelius Pass Road
Hillsboro, OR 97124
Office: 888.627.9957
Fax: 503.520.9618
www.acumed.net

These materials contain information about products that may or may not be available in any particular country or may be available under different trademarks in different countries. The products may be approved or cleared by governmental regulatory organizations for sale or use with different indications or restrictions in different countries. Products may not be approved for use in all countries. Nothing contained on these materials should be construed as a promotion or solicitation for any product or for the use of any product in a particular way which is not authorized under the laws and regulations of the country where the reader is located. Specific questions physicians may have about the availability and use of the products described on these materials should be directed to their particular authorized Acumed distributor. Specific questions patients may have about the use of the products described in these materials or the appropriateness for their own conditions should be directed to their own physician.