ACUMED®

ACUMED STABLELOC FIXATOR PINS

MediMark® Europe Sarl.
11 rue Emile ZOLA. BP 2332
38033 GRENOBLE CEDEX 2
FRANCE
+33 4 76 86 43 22

Acumed®
5885 NW Cornelius Pass Road
Hillsboro, OR 97124-9432
+1 (503) 627-9957
www.acumed.net

PKGI-28-B
Effective 11/2009

STERILE R

REF

LOT

0473

* P K G I - 2 8 - B *

OBSOLETE
Visit www.acumed.net/itd
for the latest version.

ACUMED STABLELOC FIKSATIONSSPINDE

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DESCRIPTION: Stableloc fixator pins are designed to be used in conjunction with the Stableloc External Fixator for fractures of the distal radius.

INFORMATION FOR USE: Physiological dimensions limit the sizes of implant appliances. The surgeon must select the type and size that best meets the patient's requirements for close adaptation and firm seating with adequate support.

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CONTRAINDICATIONS: Active or latent infection. Osteoporosis, insufficient quantity or quality of bone/soft tissue. Material sensitivity. If suspected, tests are to be performed prior to implantation. Sepsis. Patients who are unwilling or incapable of following postoperative care instructions. This device is not intended for screw attachment or fixation to the posterior elements (pedicles) of the cervical, thoracic, or lumbar spine.

WARNINGS: For safe and effective use of this implant, the surgeon must be thoroughly familiar with the implant, the method of application, instruments, and the recommended surgical technique for this device. The device is not designed to withstand the stress of weight bearing, load bearing, or excessive activity. Device breakage or damage can occur when the implant is subjected to increased loading associated with delayed union, nonunion, or incomplete healing. Improper insertion of the device during implantation can increase the possibility of loosening and migration. The patient must be cautioned, preferably

in writing, about the use, limitations, and possible adverse effects of this implant including the possibility of the device failing as a result of loose fixation and/or loosening, stress, excessive activity, or weight bearing or load bearing, particularly if the implant experiences increased loads due to delayed union, nonunion, or incomplete healing. The patient must be warned that failure to follow postoperative care instructions can cause the implant and/or treatment to fail.

PRECAUTIONS: An implant shall never be reused. Previous stresses may have created imperfections which can lead to device failure. Instruments shall be inspected for wear or damage prior to usage. Protect implant appliances against scratching and nicking. Such stress concentrations can lead to failure.

ADVERSE EFFECTS: Fracture of the implant due to excessive activity, prolonged loading upon the device, incomplete healing, or excessive force exerted on the implant during insertion. Implant migration and/or loosening. Metal sensitivity or histological or allergic reaction resulting from implantation of a foreign material. Pain, discomfort, or abnormal sensations due to the presence of an implant. Nerve damage resulting from surgical trauma. Necrosis of bone or bone resorption. Necrosis of tissue or inadequate healing.

STERILITY: This product is provided presterile and was exposed to a minimum dose of 25.0 kGy gamma irradiation. Restereilization may only be performed if the original sterile package has been opened in error using one of the following methods:

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Visit www.acumed.com for the latest version.

ACUMED STABLELOC FIXATOR PINS

FOR THE PERSONAL ATTENTION OF THE OPERATING SURGEON

DESCRIPTION: The Acumed Stableloc Fixator Pins are designed to be used in conjunction with the Stableloc External Fixator for fractures of the distal radius.

INFORMATION FOR USE: Physiological dimensions limit the sizes of implant appliances. The surgeon must select the type and size implant that best meets the patient's requirements for close adaptation and firm seating with adequate support.

INDICATIONS: Used in conjunction with the Stableloc External Fixator to address fracture reduction and alignment in the distal radius.

CONTRAINDICATIONS: Active or latent infection. Osteoporosis, insufficient quantity or quality of bone/soft tissue. Material sensitivity. If suspected, tests are performed prior to implantation. Sepsis. Patients who are unwilling or incapable of following postoperative care instructions. This device is not intended for screw attachment or fixation to the posterior elements (pedicles) of the cervical, thoracic, or lumbar spine.

WARNINGS: For safe effective use of this implant, the surgeon must be thoroughly familiar with the implant, the methods of application, instruments, and the recommended surgical technique for this device. The device is not designed to withstand the stress of weight bearing, load bearing, or excessive activity. Device breakage or damage can occur when the implant is subjected to increased loading associated with delayed union, nonunion, or incomplete healing. Improper insertion of the device during implantation can increase the possibility of loosening or migration. The patient must be cautioned, preferably

in writing, about the use, limitations, and possible adverse effects of this implant including the possibility the device failing as a result of loose fixation and/or loosening, stress, excessive activity, or weight bearing or load bearing, particularly if the implant experiences increased loads due to delayed union, nonunion, or incomplete healing. The patient must be warned that failure to follow postoperative care instructions can cause the implant and/or treatment to fail.

PRECAUTIONS: An implant shall never be reused. Previous stresses may have created imperfections, which can lead to a device failure. Instruments shall be inspected for wear or damage prior to usage. Protect implants against scratching and nicking. Such stress concentrations can lead to failure.

ADVERSE EFFECTS: Fracture of the implant due to excessive activity, prolonged loading upon the device, incomplete healing, or excessive force exerted on the implant during insertion. Implant migration and/or loosening. Metal sensitivity or histological or allergic reaction resulting from implantation of a foreign material. Pain, discomfort, or abnormal sensations due to the presence of an implant. Nerve damage resulting from the surgical trauma. Necrosis of bone or bone resorption. Necrosis of the tissue or inadequate healing.

STERILITY: This product is provided sterile was exposed to a minimum dose of 25.0 kGy gamma irradiation. Resterilization of a sterile device may only be performed if the original sterile package has been opened in error using one of the following methods below.

Sterilization Methods

Gravity displacement autoclave	270° F (132° C) for 30 minutes
Pre-vacuum autoclave	270° F (132° C) for 4 minutes

- Please consider your sterilization equipment manufacturer's written instructions for the specific sterilizer and load configuration used.
- Follow current AORN "Recommended Practices for Sterilization in Perioperative Practice Settings" and ANSI/AAMI ST79: 2006 – Comprehensive guide to steam sterilization and sterility assurance in health care facilities.
- Flash sterilization is not recommended, but if used, should only be performed according to requirements of ANSI/AAMI ST79: 2006 – Comprehensive guide to steam sterilization and sterility assurance in health care facilities.

STORAGE INSTRUCTIONS: Store in a cool dry place and keep away from direct sunlight. Prior to use, inspect product package for signs of tampering, or water contamination. Use oldest lots first.

CAUTION: Federal Law (USA) restricts this product sale by or on the order of a physician or hospital.

Visit www.acumed.net/lifu for the latest version.